

# RejûvaSil®

## Silicone Scar Gel

**Product Description:** RejûvaSil® Silicone Scar Gel is a topical liquid silicone gel, made with a proprietary formula designed to dry quickly and leave a thin, transparent film that does not flake off on the skin.

**Active Ingredients:** cyclomethicone, dimethicone crosspolymer, tetrahexyldecyl ascorbate, Emu oil, and squalene.

**Product Indications:** RejûvaSil® Silicone Scar Gel is used to improve the appearance of surgical and traumatic scars (including normal post-op scars, hypertrophic scars and keloid scars) and burns. It is applied to a clean and dry scar up to 3 times daily. RejûvaSil® dries to a thin semi-occlusive film on the skin promoting a moisture barrier. This barrier supports the mechanism of action, which includes hydration.

### Method of application

- Apply a thin layer of the gel to clean and dry scars, 2-3 times daily.
- Allow to dry before applying makeup or sunscreen.
- Use for at least 2-3 months, until scar maturity.
- Child usage should always be under adult supervision.

**Contraindications:** It cannot be used for open wounds. It can only be used externally.

Avoid contact with eyes and mucosa. If discomfort or rash occurs, discontinue use and consult the Doctor.

If any side effect occurs, please consult your physician or pharmacist immediately.

**Storage condition:** Store below 25 °C in a dry place. Keep out of reach of Children.

**Shelf life:** 36 months from manufacturing date.

### Manufacturer

Atlantic Medical Products, LLC

13191 Starkey Rd, Bldg 11

Largo, FL 33773, USA

[www.rejuvaskin.com](http://www.rejuvaskin.com)

10000 IFU 20-06-20